TECOMET

beere medical®

EN	Reprocessing Surgical Instruments	SV	Ombearbetning av kirurgiska instrument
FR	Retraitement des instruments chirurgicaux	IS	Fjölnota skurðáhöld
DE	Aufbereitung von chirurgischen Instrumenten	BG	Обработка на хирургически инструменти за повторна употреба
ES	Reprocesamiento de instrumentos quirúrgicos	HR	Reprocesuiranje kirurških instrumenata
IT	Riciclaggio degli strumenti chirurgici	SR	Репроцесовање хируршких инструмената
PT	Instrumentos cirúrgicos de reprocessamento	SL	Priprava kirurških instrumentov za ponovno uporabo
CS	Opakované zpracování chirurgických nástrojů	RO	Reprocesarea instrumentelor chirurgicale
DA	Genforarbejdning af kirurgiske instrumenter	RU	Повторная обработка хирургических инструментов
NL	Hersterilisatie van chirurgische instrumenten	UK	Повторне оброблення хірургічних інструментів
ET	Kirurgiliste instrumentide töötlemine	KK	Тазартылган хирургия құралдары
FI	Kirurgisten instrumenttien uudelleenprosessointien	TK	Cerrahi Aletlerin Tekrar İşlenmesi
EL	Обработка на хирургически инструменти за повторна употреба	AR	إعادة معالجة الأدوات الجراحية
HU	Sebészi műszerek újrafeldolgozása	TC	外科儀器再處理
LV	Kirurgisko instrumentu atkārtota apstrāde	SC	外科器械再处理
LT	Chirurginių instrumentų apdorojimo procesas	KO	수술 기구 재처리
NO	Gjenbruk av kirurgiske instrumenter	ID	Memroses Kembali Peralatan-Peralatan Operasi
PL	Sterylizacja instrumentów chirurgicznych	JP	手術器具の再処理
SK	Opakované spracovanie chirurgických inštrumentov		

Meditec Source
GmbH & Co. KG
Satterstraße 19
78532
Tuttlingen, Germany

© Copyright 2013 Tecomet

Tecomet
Kenosha, WI 53144 USA
Toll Free 855-925-4500
262-925-4500

KLJHFU0002-06

Reprocessing Surgical Instruments

Issue Date: 2021/11/10

Product

Surgical instruments

Warning Notice

Instruments are provided non-sterile. Clean and sterilize prior to each use. Disassemble instruments with removal parts prior to cleaning and sterilization. During cleaning of tight lumina or drill holes special attention might be required. **USA – RX ONLY – Caution: Federal (USA) law restricts this device to sale by or on the order of a licensed healthcare practitioner.**

Advise

Devices which cannot be reprocessed by the following processes are labeled accordingly. Individual procedures accompanying those devices must be followed. In case of damage, the device should be reprocessed before returning to the manufacturer for repair.

Reprocessing Limitations:

Repeated reprocessing has minimal effect upon manual instruments. End of life is determined by wear and damage due to use.

Preparation at the Point of Use:

Remove gross soiling by submerging the instrument into cold water (<40°C) immediately after use.

Don't use a fixating detergent or hot water (>40°C) as this can cause the fixation of residua which may influence the result of the reprocessing process. Remove surface contamination with paper tissue.

Containment / Transportation:

Follow hospital protocols when handling contaminated and

bio-hazardous materials. Instrument should be cleaned within 30 minutes after use to minimize the potential of staining, damage, and drying.

Preparation for Decontamination:

If possible, the devices must be reprocessed in a disassembled or opened state.

Pre-Cleaning:

Not required.

Cleaning:

Manual Cleaning Process:

- Rinse under running tap water (<40°C) until all visible soil has been removed. If needed, a soft bristle brush should be used to remove visible soil.
- Submerge instruments in an enzymatic detergent (if ultrasonic bath is used, ultrasonic process of 3 minutes and ultrasonic frequency of 35 kHz have been shown to be effective). Follow detergent manufacturer's instructions.
- Rinse the instrument under running tap water to remove the detergent.

Drying:

Manual Drying:

- Dry the instrument with a lint-free towel. The instrument may never be heated up >140°C. To avoid water residues, insufflate cavities of instruments by using sterile compressed air.

Inspection / Functional Testing:

1. Carefully inspect each device to ensure that all visible blood and soil has been removed.
2. Visually inspect for damage and / or wear.

3. Check the action of moving parts (such as hinges and box-locks) to ensure smooth operation throughout the intended range of motion.
4. Check instruments with long slender features (particularly rotating instruments) for distortion.
5. Where instruments form part of a larger assembly, check that the devices assemble readily with mating components.

Note: If damage or wear is noted that may compromise the function of the instrument, do not use the instrument, and notify the appropriate responsible person.

Packaging:

Appropriate packaging for sterilization.

Sterilization:

Sterilize instruments by applying a fractionated pre-vacuum process under consideration of the respective country requirements.

We have validated and recommend using the following parameters for the fractionated pre-vacuum process or gravity-process: 132°C (270°F) for 10 minutes and 20 minutes drying time, 121°C (250°F) for 30 minutes and 15 to 30 minutes drying time.

Flash sterilization is not allowed on lumen instruments!

Warning:

Tecomet recommends that these devices are not compatible with automated cleaning processes.

Storage:

Storage of sterilized instruments in a dry, clean, and dust-free environment at modest temperatures of 5°C to 40°C.

Maintenance:

Lubricate hinges, threads and other moving parts with a commercial water-based surgical grade instrument lubricant (such as instrument milk) to reduce friction and wear. Follow lubricant manufacturer's instructions.

Additional Advices:

It is the duty of the user to ensure that the reprocessing processes are followed; that resources and materials are available to capable personnel; and that hospital protocols are followed. State of the art and often national law requiring these processes and included resources to be validated and maintained properly.

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